Dedicated to health **ADE**

M118000 / M118000-01

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## 11. Electromagnetic compatibility

Electrical equipment is subject to special precautions regarding EMC and must be installed and commissioned in accordance with the guidance below.

Portable and mobile HF devices (e.g. mobile phones) may affect medical electrical equipment.

The use of third-party accessories may increase the emission or reduce the immunity of the device. Do not use mobile phones or similar devices that emit electromagnetic fields near the product. This could adversely affect the functionality of the product.

### Guidance and manufacturer's declaration - electromagnetic emissions

This PRODUCT is intended for use in the electromagnetic environment specified below. The customer or the user of the PRODUCT should make sure that it is used in such an environment.

Emission tests	Compliance	Electromagnetic environment – guidelines
HF emissions as per CISPR 11/EN55011	Group 2	The device must emit electromagnetic energy to perform its intended function. Electronic devices in the area can be affected.
HF emissions as per CISPR 11/EN55011	Class B	The PRODUCT is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage network that supplies buildings used for domestic purposes.
Harmonic emissions as per IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions as per IEC 61000-3-3	Complies	

### Guidance and manufacturer's declaration - electromagnetic immunity

This PRODUCT is intended for use in the electromagnetic environment specified below. The customer or the user of the PRODUCT should make sure that it is used in such an environment.

Immunity tests	IEC 60601 Test Level	Compliance level	Electromagnetic environment – guidelines
Electrostatic discharge (ESD) according to IEC 61000-4-2	± 8 kV contact discharge ± 15 kV air discharge	± 8 kV contact discharge ± 15 kV air discharge	Floors should be made of wood or concrete, or covered with ceramic tiles. If the floor is covered with synthetic material, the relative air humidity must be at least 30%.
Electrical fast transient/ burst as per IEC 61000-4-4	± 2 kV power lines ± 1 kV for input and output lines	± 2 kV power lines ± 1 kV for input and output lines	The quality of the supply voltage should correspond to that of a typical business or hospital environment.
Surges according to IEC 61000-4-5	± 1 kV conductor-conductor ± 2 kV conductor-earth	± 1 kV conductor-conductor ± 2 kV conductor-earth	The quality of the supply voltage should correspond to that of a typical business or hospital environment.
Voltage drops, short-term interruptions, and fluctuations of the supply voltage according to IEC 61000-4-11	0% UT for 1/2 period (100% break-in) 0% UT for 1 period (100% break-in) 40% UT for 5 periods (60% break-in) 70% UT for 25 periods (30% break-in) 80% UT for 250 periods (20% break-in) 0% UT for 250 periods (short interruption)	0% UT for 1/2 period (100% break-in) 0% UT for 1 period (100% break-in) 40% UT for 5 periods (60% break-in) 70% UT for 25 periods (30% break-in) 80% UT for 250 periods (20% break-in) 0% UT for 250 periods (short interruption)	The quality of the supply voltage should correspond to that of a typical business or hospital environment. If the user of the PRODUCT requires continued operation even during power interruptions, it is recommended that the PRODUCT be supplied by an uninterruptible power source or a battery.

Magnetic field at the supply frequency (50/60 Hz) according to IEC 61000-4-8	30 A/m	30 A/m	The line-frequency magnetic fields should correspond to the characteristics of a typical installation site in a commercial or clinical environment.
--	--------	--------	---

NOTE: UT is the alternating mains voltage prior to the application of the testing level.

#### Guidance and manufacturer's declaration - electromagnetic immunity

This PRODUCT is intended for use in the electromagnetic environment specified below. The customer or the user of the PRODUCT should make sure that it is used in such an environment.

Immunity Tests	Immunity Tests	Immunity Tests	Immunity Tests
HF conducted disturbances as per IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the PRODUCT, including cables, than the recommended separation distance calculated from the equation applicable to the transmitter frequency. Recommended protective distance: $d = 1.2\sqrt{P}$
HF radiated disturbances as per IEC 61000-4-3	10 V/m 80 MHz to 2.7 GHz	10 V/m	$d = \frac{6}{E}\sqrt{P}$ $d = 1.2\sqrt{P} \text{ 80 Mhz to 800 Mhz}$ $d = 2.3\sqrt{P} \text{ 800 MHz to 2.7 GHz}$ Where (P) is the maximum output power of the transmitter in watts (W) according to the transmitter manufacturer's specifications and d is the recommended separation distance in metres (m). The field strength of fixed RF transmitters, as determined by an electromagnetic site survey a should be less than the compliance level in each frequency range.b In the vicinity of devices that bear the following pictorial symbol, interference is possible: 

COMMENT 1 At 80 MHz and 800 MHz, the higher value shall apply.

COMMENT 2 This guidance may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- a The field strength of fixed transmitters such as base stations for wireless telephones and mobile land radio services, amateur radio stations, AM and FM radio and television transmitters cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed HF transmitters, an electromagnetic site survey should be considered. If the field strength measured at the location of the PRODUCT exceeds the compliance level indicated above, the PRODUCT should be monitored for its normal operation at each place of use. If abnormal performance characteristics are observed, additional measures may be necessary such as reorienting or relocating the PRODUCT.
- b In excess of the frequency range 150 kHz to 80 MHz, the field strength should be less than 3 V/m.

---

**Recommended separation distances between portable and mobile HF communications equipment and the PRODUCT**

---

The PRODUCT is intended for operation in an electromagnetic environment in which HF radiated disturbances are monitored. The customer or user of the PRODUCT can help prevent electromagnetic interferences by maintaining minimum distances between portable and mobile HF communications equipment (transmitters) and the PRODUCT as recommended below, according to the maximum output power of the communications equipment.

---

Nominal power of the transmitter (W)	Separation distance according to transmitter frequency (m)		
	150 KHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 MHz to 2.7 GHz $d = 2,3\sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23

F For transmitters whose maximum output power is not listed in the table above, the distance can be determined using the equation specified in the respective column, where P is the maximum output power of the transmitter in watts (W) according to the transmitter manufacturer's specification.

COMMENT 1	An additional factor of 10/3 has been used to determine the recommended separation distance of transmitters in the frequency range of 80 MHz to 2.7 GHz in order to reduce the likelihood that a mobile/portable communication device placed inadvertently in the patient area will result in interference.
COMMENT 2	This guidance may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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#### **ADE Germany GmbH**
Manufacturer: ADE Germany GmbH
Knorring Oy Ab